



# Elbow/Bicep Wrap

*2000 Series*

## *Cold Compression Therapy Wrap*




T-2000KBS

SEGMENTAL

NON-STERILE

### Instructions:

- Attach each elastic strap to the right side of each wing.
- Secure each elastic strap around the wrap (see photo 2) to create a cylinder.
- Insert the injured area inside the wrap as shown in the pictures.
- Readjust the straps to form a snug, secure fit around the injured area.
- Connect the hose from the wrap to the machine. The connectors will make a “click” sound when connected properly.

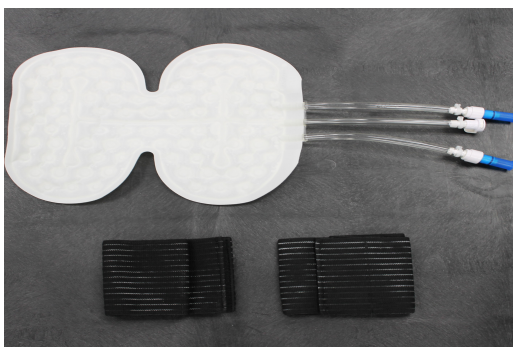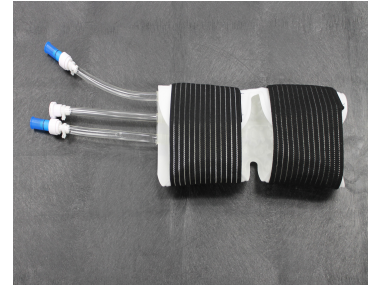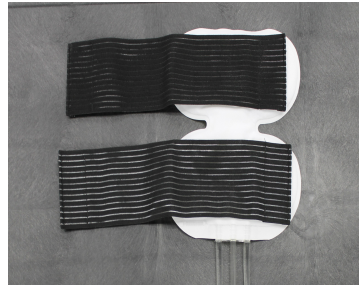

Questions? Please call  
734-492-5962

**Precautions:**

Carefully read user instructions and warnings prior to operation.

Therapy cuffs are non-sterile and should not be directly applied to an open wound. Do not use directly over broken skin. This product should not be used when you have any of the following active conditions: inflammatory phlebitis process, episodes of pulmonary embolism, congestive heart failure, pulmonary edema, suspected deep vein thrombosis, acute inflammations of the veins (thrombophlebitis), decompensated cardiac insufficiency, arterial dysregulation, erysipelas, deep acute venal thrombosis (phlebothrombosis), carcinoma and carcinoma metastasis in the affected extremity, decompensated hypertonia, acute inflammatory skin diseases, infection, venous or arterial occlusive disease or in any instances where increased venous and lymphatic return is undesirable. Use with caution on any limb which is numb or not sensitive to pain. If you have extremely low blood pressure, check with your doctor before using these products.

A licensed healthcare practitioner must select the correct temperature setting; patients vary in sensitivity to cold. A regular check of the temperature must be made once it has been established for you. Caution should be taken during prolonged use, for children, diabetics, incapacitated patients, and those with decreased skin sensitivity or poor circulation. Due to individual differences in sensitivity and susceptibility to cold, check your skin frequently. Follow instructions of your physician for length, frequency and duration of treatment.

These products are to be fitted initially by a healthcare professional familiar with the purpose for which they are used. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare professionals, care providers involved in your care, and you. If unusual swelling, skin discoloration or discomfort occurs, discontinue use, contact the representative and/or the person who prescribed use of this equipment.

A therapy cuff with therapy unit may be used in a medical facility or clinical environment with direct healthcare provider supervision. If the prescribing healthcare practitioner determines it is appropriate for you to use the therapy cuff at home, the healthcare practitioner must provide you with adequate and appropriate instructions for how to use the device. Further, the healthcare practitioner must monitor your use of the device to assure that you are using the equipment properly.

Cold therapy should not be used if you have Raynaud's Disease, poor peripheral circulation, diabetes, decreased skin sensitivity or hypersensitivity to cold.

Do not attempt to sterilize a non-sterilized cuff by any means.

Do not use cuff near open flame.

Do not smoke while therapy cuffs are in use.

Disposable therapy cuffs are designed for single patient use only and may be used on the same patient for the length of the treatment.

Do not wrap the therapy cuff so tightly as to restrict blood or fluid flow. Do not use pins to secure the therapy cuff or hoses.

Do not allow the therapy cuff or hoses to contact sharp objects.

Dressings used under the therapy cuff should be applied lightly.

Cuffs should not be left open (off of the patient) and fully inflated for extended periods of time.

**Warranty Information:** WMI Enterprises Products

WMI Enterprises warrants the product in accordance with the following terms: *Full Replacement:* Products manufactured or supplied by WMI shall be free from defects in material and workmanship upon receipt of the product and for the initial use of the product. WMI shall replace the product, at no charge to the customer, if in the reasonable judgment of WMI, the product is found to have material or workmanship faults. The product must be returned to WMI for warranted evaluation. The buyer shall pay for the expense of returning the product to WMI for warranty service. Return products must be in the WMI approved box and packing material. WMI will pay for the expense of returning the product back to the buyer. The buyer shall be responsible and may be assessed a fee for testing if no defects are found with the product. *In the event that the product or any portion thereof is not installed or used in accordance with the manufacturer's Operating Instructions, any and all warranties either expressed or implied shall be and are hereby voided.* Only upon the proper installation and use of the items shall this warranty apply. This warranty does not cover any product that has been subject to misuse, neglect or accident. This warranty does not cover any product that has been altered or modified so as to change its intended use. In addition, this warranty does not extend to the use of fluids which are either incompatible with the unit or adversely affect its operation, performance, or durability. Because WMI constantly provides our customers with the latest technology, we reserve the right to change or improve the design of any product without assuming any obligation to modify any product previously manufactured.

**Disclaimer**

The information contained in this document is provided "as is". WMI expressly disclaims all information including, but not limited to, express and implied warranties of merchantability, fitness for a particular use, or non-infringement. In no event will WMI be liable for any direct, indirect, special, incidental or consequential damages, including lost profits, lost business or lost data, resulting from the use of or reliance upon the information, whether or not WMI has been advised of the possibility of such damages.

WMI assumes no responsibility for the accuracy or completeness of the information presented which is subject to change without notice.